

“Simpl” Implant Abutments with Atlantis CAD CAM Technology



**Don't change your implant...
change your abutment.**

**See more patients in less time...
and save money in the process!**

See how we've made it "Simpl."



Making Implant Restorations “Simpl”

CAD CAM Abutments

Simplified Protocol



Making Implant Restorations “Simpl”

- Atlantis CAD CAM Abutments reduce chair time, promote tissue healing and guarantee fit.
- Town & Country has incorporated these abutments into a complete restorative package, called “Simpl.”
- “Simpl” makes it easy for you to plan, price and place implant restorations.



Making Implant Restorations “Simple”



Atlantis CAD CAM Abutments are compatible with virtually every implant system, including:

- 3i Micro, Miniplant, Standard, Wide, Certain
- Astra ST
- BioHorizons
- Innova Endopore
- Lifecore
- Nobel Biocare, Branemark, SteriOss Replace, Select
- Sterngold-ImplaMed
- Straumann RN
- Zimmer Screw-Vent, Spline, SwissPlus
- and others

Making Implant Restorations “Simpl”

0 Parts, 1 Price, 2 Weeks



Our “Simpl” package includes:

- Soft tissue model
- Analog
- Custom-milled Titanium Abutment
- Titanium abutment screw
- Abutment placement jig
- **One price of just \$379 per unit**

PLUS...

- Add the crown of your choice

Making Implant Restorations “Simple”

Add the crown of your choice



PFM:

- Phoenix PFM
- NobleYellow, Premium Noble
- Captek, High Noble
- High Noble White
- High Noble Yellow
- Semi-Precious


All-Ceramic:

- Procera Alumina
- Procera Zirkon
- 3M Lava Zirconia
- Cercon Zirconia

Making Implant Restorations “Simpl”
CAD CAM Abutments
Simplified Protocol




What's a CAD CAM Abutment?

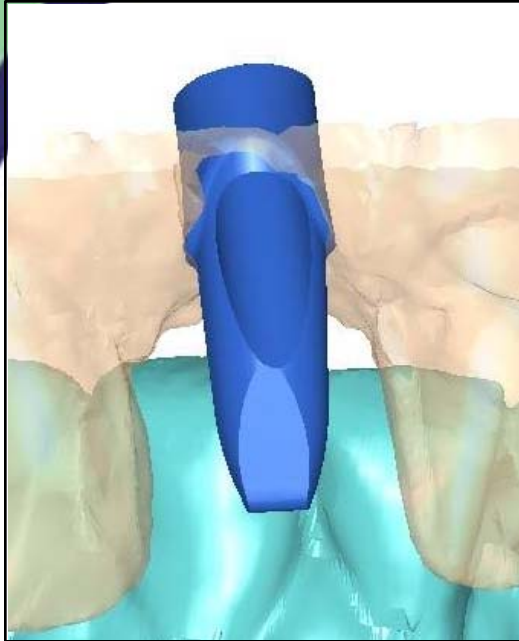

- Designed by computer, reviewed and approved by Town & Country senior technician
- Milled by computer for unsurpassed accuracy and precision
- Cut from medical-grade Titanium providing superior biocompatibility and best possible integration with implant fixture
- Over 40,000 abutments fabricated to-date

What's a CAD CAM Abutment?


- Precision Milled from Titanium
- Perfect Emergence Profile
- Angled Implant Axis

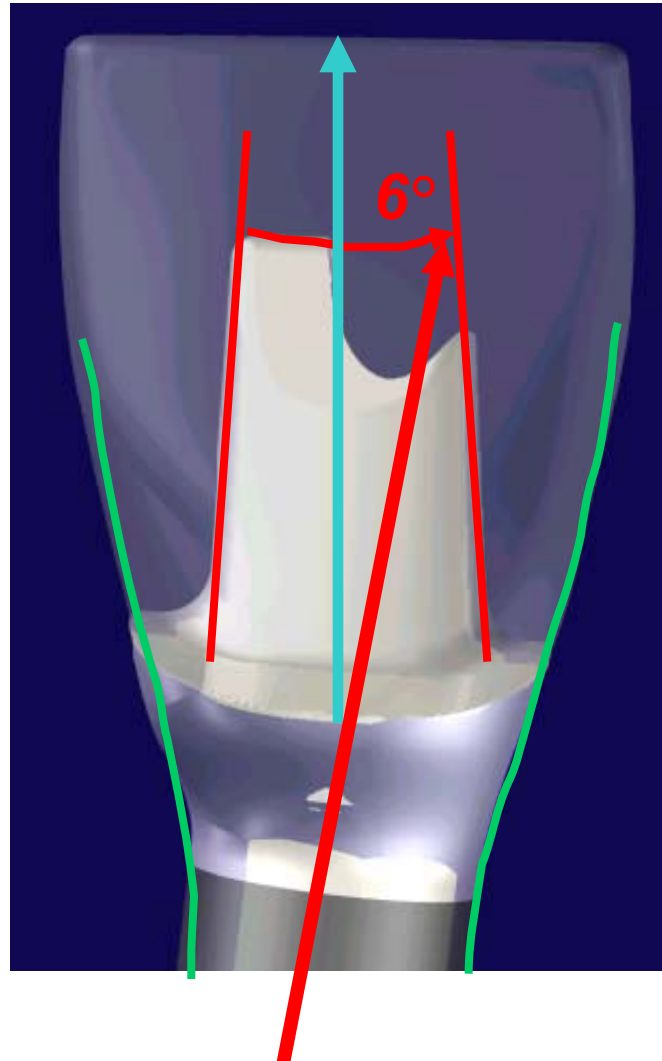

- Straight Path of Insertion
- Correct Path of Insertion
- Ideal Coronal Preparation

What's a CAD CAM Abutment?

- Shaped like a natural tooth
- Ideal emergence profile
- Compensates for clinical issues (angulation, reduced occlusal height)
- No stock abutments to inventory
- Reduce chair time to two restorative patient visits

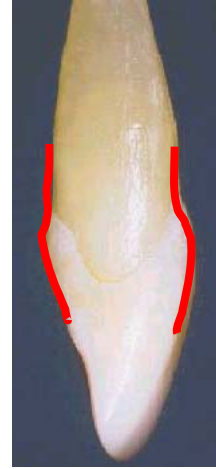

Angled Implant Axis



What's a "GEMINI" Abutment?



Computer
Design



Primary
Abutment



"Gemini"
Duplicate

A functionally identical abutment that decreases chair time for both surgeon and referring dentist.

Making Implant Restorations “Simpl”
CAD CAM Abutments
Simplified Protocol



Simplified Protocol

Restorative Consult

- Take pre-op models for transfer jig and provisional
- Refer patient to your surgeon for a Simpl Implant using Atlantis Abutments

Surgical Visit #1

- Place implant; take fixture level index
- Send index and case materials to Town & Country



- Design and machine Atlantis Titanium Abutment
- Return case to surgeon

Surgical Visit #2

- Place Atlantis Abutment and provisional

Restorative Visit #1

- Check tissue healing
- T&C makes final crown on Gemini duplicate abutment

Restorative Visit #2

- Deliver final restoration

Simplified Protocol

Pre-Operative Restorative Consultation

- Pre-Operative Full-Mouth Impression
- Opposing model
- Bite Registration
- Shade Selection
- Specify temporization of choice



Simplified Protocol

Surgical Visit 1

- Place Implant
- Take Fixture Level Index
- Transfer jig available



Simplified Protocol *Between Surgical Visits 1 and 2*



Atlantis creates a custom patient-specific abutment.



Insert provisional crown of your choice.

Simplified Protocol

Surgical Visit 2

Place abutment



Cement provisional



Suture soft tissue



Simplified Protocol

Restorative Visit 1

Check tissue health



Send Gemini to T&C for final crown



No Final Impression!



Simplified Protocol Restorative Visit 2

*Remove
provisional*



*Cement final
restoration*



Final result



Why make your implants “Simpl”?

Easy to plan – no parts required.

Easy to price – just \$379 plus crown.

Easy to place – just 2 restorative visits.

CAD CAM delivers unsurpassed accuracy and fit.

Titanium is the most bio-compatible material.

**Ideal for all single, bridge and
full-mouth reconstruction cases.**

**Surgeons use Atlantis Abutments
instead of healing caps.**

Why Town & Country?

We are a leading 43-year-old crown & bridge dental lab.

**The first Atlantis “Elite” Laboratory
in the New York metropolitan area.**

**We have the confidence and expertise to
handle any case...implant or otherwise.**

**We are fabricating close to 100 Atlantis units per month.
No other laboratory comes close.**





TOWN & COUNTRY
A better dental lab

Better Products • Better Service • Better Advice

www.BetterDentalLab.com

1-800-925-8696

